 **BD BBL™ Sensi-Disc™**
Single Disc Dispenser

8860321JAA
2010/10

Cat. No. 260457
260459

Pokyny vám poskytne miestni zástupce spoločnosti BD. / Kontakt den lokale BD repræsentant for at få instruktioner. / Kasutusjuhiste suhtes kontakteeruge oma kohaliku BD esindajaga. / Επικοινωνήστε με τον τοπικό αντιπρόσωπο της BD για οδηγίες. / A használati utasítást kérje a BD helyi képviselőjétől. / Naudojimo instrukcijų teiraukitės vietos BD įgaliotojo atstovo. / Kontakt din lokale BD-representant for mer informasjon. / Aby uzyskać instrukcje użytkowania, skontaktuj się z lokalnym przedstawicielstwem BD. / Contacte o seu representante local da BD para obter instruções. / Instrukcie získate u miestneho zástupcu spoločnosti BD. / Kontakta lokal Vecton Dickinson-representant för anvisningar. / Свяжитесь с местным представителем на BD за инструкции. / Contactați reprezentantul dumneavoastră local BD pentru instrucțiuni. / Talimatlar için yerel BD temsilcilerinize danışın. / Obratite se svom lokalnom predstavniku kompanije BD za uputstva. / Для получения инструкций свяжитесь с местным представителем компании BD. / Өзіңіздің жергілікті BD өкіліне жүгініп нұсқау алыңыз. / Kontaktiraj lokalnog predstavnika BD za upute.

INTENDED USE

The Sensi-Disc™ Single Disc Dispenser is recommended for the individual application of cartridge BBL™ brand antimicrobial susceptibility test and biochemical differentiation discs onto the surface of inoculated plated media.

The Single Disc Dispenser replaces the manual placement of discs using sterile forceps.

Precautions: For *in vitro* Diagnostic Use.

Observe aseptic techniques and established precautions against microbiological hazards throughout all procedures.

PRODUCT DESCRIPTION

Single Disc Dispensers are made of durable 24 gauge stainless steel. Store the *loaded* dispenser in the refrigerator at 2 – 8°C when not in use.

INSTRUCTIONS

Load a dispenser by placing the bottom of a cartridge into the bottom of the dispenser, then press the top of the cartridge into the top of the dispenser where it will be retained by a friction fit.

Operate the loaded unit by pressing the spring steel arm which will eject one disc.

BD BBL Sensi-Disc Single Disc Dispenser

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 **BD BBL Sensi-Disc**
Single Disc Dispenser

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 **BD BBL Sensi-Disc
Single Disc Dispenser**

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Becton, Dickinson and Company © 2010 BD.
BD, BD Logo, BBL and Sensi-Disc are trademarks of
Benex Limited