
Revisions

SO 0191-5

Rev from	Rev to	ECO #
0703	2010/06	5348-10

Notes:

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
- BD Cat. Number 261188
- Blank (Sheet) Size: Length: 11" Width: 18"
 Number of Pages: 8 Number of Sheets: 1
 Page Size: Length 11" Width 4.5" Final Folded Size: 4.5" x 1 7/8"
- Style (see illustrations below): # 4


#1

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
#2


#3

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
#4
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
#5
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
#6
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
#7

- See Specification Control Number L001194 for Material Information
- Ink Colors: Printed two sides ☒ Yes ☐ No
 No. of Colors: 1 PMS# 2755 Blue
- Graphics are approved by Becton, Dickinson and Company. Supplier has the responsibility for using the most current approved revision level

Label Design	Date	COMPANY CONFIDENTIAL. THIS DOCUMENT IS THE PROPERTY OF BECTON, DICKINSON AND COMPANY AND IS NOT TO BE USED OUTSIDE THE COMPANY WITHOUT WRITTEN PERMISSION	 Becton, Dickinson and Company 7 Loveton Circle Sparks, MD 21152 USA
Proofer	Date		
Checked By	Date		
Part Number: L001194	Category and Description Package Insert, Lactophenol Cotton Blue Stain Droppers	Sheet: 1 of 9	A
		Scale: N/A	

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INTENDED USE

BBL™ Lactophenol Cotton Blue Stain Droppers are intended for use as a stain in the examination of fungi.^{1,2}

SUMMARY AND EXPLANATION

The solution can be used in wet mounts in the examination of yeasts and molds and serves as both a mounting fluid and a stain.^{1,2}

PRINCIPLES OF THE PROCEDURE

Lactophenol cotton blue is a blue stain used for direct examination of clinical specimens for fungal elements. Upon the addition of lactophenol cotton blue, fungi stain blue allowing for easier visualization and examination.

REAGENTS

BBL Lactophenol Cotton Blue Stain Droppers contain 0.5 mL of lactophenol cotton blue solution consisting of glycerol 40%, phenol 20%, lactic acid 20%, cotton blue dye (Poirrier's blue) 0.05%, and water.

Warnings and Precautions:

For *in vitro* Diagnostic Use.

Lactophenol cotton blue is acidic. Avoid contact with the skin. Rinse thoroughly with water if spilled.

Follow established laboratory procedures in handling and disposing of infectious materials.

Storage Instructions: Store at controlled room temperature 15 – 30°C (59 – 86°F). Protect from light.

Product Deterioration: Reagents are hermetically sealed in an ampule, which affords protection of the solution from chemical instability until expiration date.

PROCEDURE

Material Provided: Lactophenol Cotton Blue Stain Droppers.

Materials Required But Not Provided: Ancillary culture media, reagents, quality control organisms and laboratory equipment as required for this procedure.

Test Procedure

1. Hold dropper upright and **POINT TIP AWAY FROM YOURSELF**. Grasp the middle with thumb and forefinger and squeeze gently to break ampule inside the dropper. **Caution: Break ampule close to its center one time only. Do not manipulate the dropper any further as the plastic may puncture and injury may occur.** Tap on tabletop a few times.
2. Invert dropper for convenient drop-by-drop dispensing.
3. Prepare a wet mount with the stain according to commonly accepted laboratory methodology, such as described in the references below.

User Quality Control

Use of the Lactophenol Cotton Blue Stain Droppers in preparing a wet mount of a mold or yeast will stain the specimen light blue and allow it to be easily visualized by microscopy.

Quality control requirements must be performed in accordance with applicable local, state and/or federal regulations or accreditation requirements and your laboratory's standard Quality Control procedures. It is recommended that the user refer to pertinent NCCLS guidance and CLIA regulations for appropriate Quality Control practices.

RESULTS

Yeast cells, mycelia and fruiting structures stain a delicate blue color, while the background appears a faint, pale blue.

LIMITATIONS OF THE PROCEDURE

Lactophenol cotton blue is only useful in the staining of yeasts and molds and when used as a mounting medium.

PERFORMANCE CHARACTERISTICS

Prior to release, all lots of **BBL** Lactophenol Cotton Blue Stain Droppers are tested to verify specific product characteristics. The identification of the cotton blue dye in Lactophenol Cotton Blue Stain is determined by spectrophotometer analysis. The percentage of lactic acid in the Lactophenol Cotton Blue Stain is determined by titration. The percentage of phenol in Lactophenol Cotton Blue Stain is determined by ultraviolet spectrophotometer analysis.

Each lot is also tested for biological performance. One drop of Lactophenol Cotton Blue Stain is placed on a microscope slide. A small amount of an *Aspergillus niger* culture (ATCC™ 16404) is added to each drop of stain. With an inoculating needle, the mycelia are gently teased apart. After 1 min the slide is examined. Fungal elements are stained a delicate blue color, while the background appears a faint, pale blue. Additional slides are prepared without a fungal specimen and upon microscopic examination, no extraneous organisms are observed.

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ATCC is a trademark of the American Type Culture Collection.
BD, BD Logo and BBL are trademarks of Becton, Dickinson and Company. ©2010 BD.
Benex Limited