



BD BBL™ Bacteroides Bile Esculin Agar (BBE)
L007348 • Rev. 08 • November 2016

QUALITY CONTROL PROCEDURES (Optional)

R_x Only

I INTRODUCTION

BD BBL™ Bacteroides Bile Esculin Agar is a selective medium for the isolation and presumptive identification of the group.

II PERFORMANCE TEST PROCEDURE

1. Reduce all BBE plates overnight at room temperature in a **BD GasPak™ EZ** anaerobic system.
2. Preparation of inocula
 - a. Prepare the anaerobe test cultures for inoculation by swabbing the growth from 48- to 72-h culture plates into tubes of reduced Chopped Meat Broth. Incubate for two days at 36 ± 1 °C.
 - b. For *Escherichia coli* and *Proteus mirabilis*, use 4- to 5-h **BD BBL™ Trypticase™** Soy Broth cultures diluted to yield 10³–10⁴ CFU/plate.
3. Inoculation of the plates
 - a. Using a volumetric pipettor or equivalent method, deliver 0.001 mL of the appropriate inoculum to the plated media samples and streak for isolation.
 - b. Include **BD BBL Trypticase** Soy Agar with 5% Sheep Blood (TSA II) plates as controls for all organisms.
4. Incubate all plates anaerobically (**BD GasPak EZ** anaerobic system) at 35 ± 2 °C.
5. Examine all inoculated plates at 48 h for amount of growth, colony size, esculin reactions and selectivity.
6. Expected Results

| Organisms | ATCC® | Recovery | Esculin Reaction |
|-------------------------------------|-------|---------------------------|---|
| * <i>Bacteroides fragilis</i> | 25285 | Fair to heavy | + (blackening) |
| * <i>Bacteroides vulgatus</i> | 8482 | Fair to heavy | +/- (may or may not produce blackening) |
| <i>Bacteroides ovatus</i> | 8483 | Fair to heavy | + (blackening) |
| <i>Bacteroides thetaiotaomicron</i> | 29741 | Fair to heavy | + (blackening) |
| * <i>Clostridium perfringens</i> | 13124 | No growth | N/A |
| <i>Escherichia coli</i> | 25922 | No growth to trace growth | N/A |
| * <i>Proteus mirabilis</i> | 12453 | No growth to trace growth | N/A |

*Recommended organism strain for User Quality Control.

NOTE: This medium is exempt from User QC testing according to CLSI M22-A3.

III ADDITIONAL QUALITY CONTROL

1. Examine plates as described under "Product Deterioration."
2. Visually examine representative plates to assure that any existing physical defects will not interfere with use.
3. Determine the pH potentiometrically at room temperature for adherence to the specification of 7.0 ± 0.2.
4. Note the firmness of plates during the inoculation procedure.
5. Incubate uninoculated representative plates at 35 ± 2 °C for 72 h and examine for microbial contamination.

PRODUCT INFORMATION

IV INTENDED USE

BD BBL Bacteroides Bile Esculin Agar (BBE) is recommended as a primary isolation medium for the selection and presumptive identification of the *B. fragilis* group.^{1,2}

V SUMMARY AND EXPLANATION

Among the most frequently encountered anaerobes in human clinical infections are members of the "*Bacteroides fragilis* group." Rapid detection and identification of these organisms is important since they have been found to be more resistant to antimicrobial therapy than other anaerobes.³ *B. fragilis* and *B. thetaiotaomicron* are the species of greatest clinical significance.³ Other species in the group include: *B. caccae*, *B. distasonis*, *B. eggerthii*, *B. merdae*, *B. ovatus*, *B. stercoris*, *B. uniformis* and *B. vulgatus*.

Frequently these pathogens occur in a mixture of microorganisms which may overgrow the primary isolation medium. Selective media, such as CDC Anaerobe 5% Sheep Blood Agar with Kanamycin and Vancomycin, have been recommended as appropriate for primary isolation.⁴ However, limited evidence for the presumptive identification of the *B. fragilis* group was provided. In 1978, Livingston et al. described a primary plating medium (BBE) which was found to provide selective recovery of the *B. fragilis* group and also evidence for presumptive identification.¹

VI PRINCIPLES OF THE PROCEDURE

BD BBL Bacteroides Bile Esculin Agar is a primary plating medium for the selective isolation and presumptive identification of the *B. fragilis* group. Selective inhibition of facultative anaerobes and most gram-negative anaerobic organisms is obtained by the presence of gentamicin and oxgall. Differentiation of the *B. fragilis* group is based on esculin hydrolysis, which produces esculetin and dextrose. The esculetin reacts with the iron salt (ferric ammonium citrate) contained in the medium to produce a dark brown to black complex that appears in the medium surrounding colonies of members of the *B. fragilis* group.

VII REAGENTS

BD BBL Bacteroides Bile Esculin Agar

Approximate Formula* Per Liter Purified Water

| | | | |
|-------------------------------------|--------|------------------------------|--------|
| Pancreatic Digest of Casein | 14.5 g | Hemin | 0.01 g |
| Papaic Digest of Soybean Meal | 5.0 g | Gentamicin | 0.1 g |
| Sodium Chloride | 5.0 g | Vitamin K ₁ | 0.01 g |
| Esculin | 1.0 g | Agar | 14.0 g |
| Ferric Ammonium Citrate | 0.5 g | Growth Factors | 1.8 g |
| Oxgall | 15.0 g | | |

*Adjusted and/or supplemented as required to meet performance criteria.

Warnings and Precautions: For *in vitro* Diagnostic Use.

If excessive moisture is observed, invert the bottom over an off-set lid and allow to air dry in order to prevent formation of a seal between the top and bottom of the plate during incubation.

Pathogenic microorganisms, including hepatitis viruses and Human Immunodeficiency Virus, may be present in clinical specimens.

"Standard Precautions"⁵⁻⁸ and institutional guidelines should be followed in handling all items contaminated with blood and other body fluids. After use, prepared plates, specimen containers and other contaminated materials must be sterilized by autoclaving before discarding.

Storage Instructions: On receipt, store plates in the dark at 2–8 °C. Avoid freezing and overheating. Do not open until ready to use.

Minimize exposure to light. Prepared plates stored in their original sleeve wrapping at 2–8 °C until just prior to use may be inoculated up to the expiration date and incubated for recommended incubation times. Allow the medium to warm to room temperature before inoculation.

Product Deterioration: Do not use plates if they show evidence of microbial contamination, discoloration, drying, cracking or other signs of deterioration.

VIII SPECIMEN COLLECTION AND HANDLING

Specimens suitable for culture may be handled using various techniques. For detailed information, consult appropriate texts.^{9,10} Specimens should be obtained before antimicrobial therapy has been administered. Provision must be made for prompt delivery to the laboratory.

IX PROCEDURE

Material Provided: BD BBL Bacteroides Bile Esculin Agar

Materials Required But Not Provided: Ancillary culture media, reagents, quality control organisms and laboratory equipment as required.

Test Procedure: Observe aseptic techniques.

The agar surface should be smooth and moist, but without excessive moisture. Media should be reduced immediately prior to inoculation by placing under anaerobic conditions for 6–24 h.¹¹ An efficient and easy way to obtain suitable anaerobic conditions is through the use of **BD GasPak EZ** anaerobic systems.

Streak the specimen as soon as possible after it is received in the laboratory. As some strains of the *B. fragilis* group may not grow well due to the selective properties of the medium, it is advisable to include a nonselective blood agar medium such as CDC Anaerobe 5% Sheep Blood Agar.

Incubate immediately under anaerobic conditions or place in a holding jar flushed with oxygen free gas(es) until sufficient plates are accumulated (but no longer than 3 h).¹² Incubation should be at 35 ± 2 °C for at least 48 h. Regardless of anaerobic system used, it is important to include an indicator of anaerobiosis such as the **BD GasPak** disposable anaerobic indicator.

User Quality Control: See "Quality Control Procedures."

Each lot of media has been tested using appropriate quality control organisms and this testing meets product specifications and CLSI standards, where relevant. As always, QC testing should be performed in accordance with applicable local, state, federal or country regulations, accreditation requirements, and/or your laboratory's standard quality control procedures.

X RESULTS

After 48 h of incubation, colonies of the *B. fragilis* group should be greater than 1 mm in diameter and appear gray, circular, entire and raised. A Gram stain should be performed to assist in the identification. Most anaerobes other than the *B. fragilis* group are inhibited. Esculin hydrolysis is indicated by a blackening of the medium around the colonies.

NOTE: If plates are to be examined after 24 h, examine quickly and reincubate under anaerobic conditions.

In order to determine the relationship to oxygen of each colony type present on anaerobic solid media, inoculate the following media:¹³

1. One anaerobe blood agar plate to be incubated anaerobically.
2. One aerobic blood agar (or chocolate agar) plate to be incubated in an atmosphere of carbon dioxide. The chocolate agar is particularly needed to distinguish nutritionally-fastidious *Haemophilus* species and other bacteria which will grow on anaerobe blood agar incubated anaerobically and on chocolate agar under increased carbon dioxide tension but which fail to grow on blood agar in the presence of carbon dioxide or in air.
3. One aerobic blood agar plate to be incubated aerobically.
4. Tubes of Enriched Thioglycollate Medium and/or Cooked Meat Medium and a tube of Peptone Yeast Extract Glucose Broth.

Incubate all cultures at 35 ± 2 °C for a minimum of 24 h and up to 7 days.

Record the relationship to oxygen as either obligate anaerobe or nonanaerobe (aerotolerant anaerobe, microaerophilic, or facultative anaerobe).¹³

Organisms failing to grow on the aerobic subculture plates may be presumed to be obligately anaerobic in terms of their oxygen requirements.

Colonies of the type(s) which prove to be obligate anaerobes can be further studied using the corresponding broth cultures.^{14,15}

XI LIMITATIONS OF THE PROCEDURE

B. vulgatus may not hydrolyze esculin.^{2,3}

For identification, organisms must be in pure culture. Morphological, biochemical, and/or serological tests should be performed for final identification. Consult appropriate texts for detailed information and recommended procedures.^{9,10,16-19}

A single medium is rarely adequate for detecting all organisms of potential significance in a specimen. It should be recognized that organisms generally susceptible to the antimicrobial agent in a selective medium may be completely or only partially inhibited depending upon the concentration of the agent, the characteristics of the microbial strain and the number of organisms in the inoculum. Organisms that are generally resistant to the antimicrobial agent should not be inhibited. Cultures of specimens grown on selective media should, therefore, be compared with specimens cultured on nonselective media to obtain additional information and help ensure recovery of potential pathogens.

XII AVAILABILITY

| Cat. No. | Description |
|----------|-------------|
|----------|-------------|

| | |
|--------|--|
| 221836 | BD BBL™ Bacteroides Bile Esculin Agar (BBE), Pkg. of 10 plates |
|--------|--|

XIII REFERENCES

1. Livingston, S.J., S.D. Kominos, and R.B. Yee. 1978. New medium for selection and presumptive identification of the *Bacteroides fragilis* group. J. Clin. Microbiol. 7:448-453.
2. Mangels, J.I. 2004. Culture media for anaerobes, p.4.3.1-p.4.3.5 In H.D. Isenberg (ed.), Clinical microbiology procedures handbook, vol. 2. American Society for Microbiology, Washington, D.C.
3. Jousimies-Somer, H.R., and S.M. Finegold. 1991. Anaerobic gram-negative bacilli and cocci, p. 538-553. In A. Balows, W.J. Hausler, Jr., K.L. Herrmann, H.D. Isenberg, and H.J. Shadomy (ed.), Manual of clinical microbiology, 5th ed. American Society for Microbiology, Washington, D.C.
4. Dowell, V.R., Jr., G.L. Lombard, F.S. Thompson, and A.Y. Armfield. 1977. Media for isolation, characterization, and identification of obligately anaerobic bacteria. CDC laboratory manual. Center for Disease Control, Atlanta.
5. National Committee for Clinical Laboratory Standards. 2001. Approved Guideline M29-A2. Protection of laboratory workers from occupationally acquired infections, 2nd ed. NCCLS, Wayne, PA.
6. Garner, J.S. 1996. Hospital Infection Control Practices Advisory Committee, U.S. Department of Health and Human Services, Centers for Disease Control and Prevention. Guideline for isolation precautions in hospitals. Infect. Control Hospital Epidemiol. 17:53-80.
7. U.S. Department of Health and Human Services. 1999. Biosafety in microbiological and biomedical laboratories, HHS Publication (CDC), 4th ed. U.S. Government Printing Office, Washington, D.C.
8. Directive 2000/54/EC of the European Parliament and of the Council of 18 September 2000 on the protection of workers from risks related to exposure to biological agents at work (seventh individual directive within the meaning of Article 16(1) of Directive 89/391/EEC). Official Journal L262, 17/10/2000, p. 0021-0045.
9. Murray, P.R., E.J. Baron, J.H. Jorgensen, M.A. Pfaller, and R.H. Tenover (ed.). 2003. Manual of clinical microbiology, 8th ed. American Society for Microbiology, Washington, D.C.
10. Forbes, B.A., D.F. Sahm, and A.S. Weissfeld. 2002. Bailey and Scott's diagnostic microbiology, 11th ed. Mosby, Inc., St. Louis.
11. Dowell, V.R., Jr., and T.M. Hawkins. 1987. Laboratory methods in anaerobic bacteriology. CDC laboratory manual. HHS Publication No. (CDC) 87-8272. Centers for Disease Control, Atlanta.
12. Martin, W.J. 1971. Practical method for isolation of anaerobic bacteria in the clinical laboratory. Appl. Microbiol. 22:1168-1171.
13. Allen, S.D., J.A. Siders, and L.M. Marler. 1985. Isolation and examination of anaerobic bacteria, p. 413-433. In E.H. Lennette, A. Balows, W.J. Hausler, Jr., and H.J. Shadomy (ed.), Manual of clinical microbiology, 4th ed. American Society for Microbiology, Washington, D.C.
14. Rodloff, A.C., P.C. Appelbaum, and R.J. Zabransky. 1991. Cumitech 5A, Practical anaerobic bacteriology. Coordinating ed., A.C. Rodloff. American Society for Microbiology, Washington, D.C.
15. Jousimies-Somer, H.R., P.H. Summanen, H. Wexler, S.M. Finegold, S.E. Garbino, and H.N. Shah. 2003. *Bacteroides*, *Porphyromonas*, *Prevotella*, *Fusobacterium*, and other anaerobic gram-negative bacteria, p. 880-901. In P.R. Murray, E.J. Baron, J.H. Jorgensen, M.A. Pfaller, and R.H. Tenover (ed.), Manual of clinical microbiology, 8th ed. American Society for Microbiology, Washington, D.C.
16. Holt, J.G., N.R. Krieg, P.H.A. Sneath, J.T. Staley, and S.T. Williams (ed.). 1994. Bergey's Manual™ of determinative bacteriology, 9th ed. Williams & Wilkins, Baltimore.
17. MacFaddin, J.F. 2000. Biochemical tests for identification of medical bacteria, 3rd ed. Lippincott Williams & Wilkins, Baltimore.
18. Koneman, E.W., S.D. Allen, W.M. Janda, P.C. Schreckenberger, and W.C. Winn, Jr. 1997. Color atlas and textbook of diagnostic microbiology, 5th ed. Lippincott-Raven, Philadelphia.
19. Isenberg, H.D. (ed.). 2004. Clinical microbiology procedures handbook, vol. 1, 2 and 3, 2nd ed. American Society for Microbiology, Washington, D.C. 258:74-79.

Technical Information: In the United States contact BD Technical Service and Support at 1.800.638.8663 or www.bd.com.

Becton, Dickinson and Company
7 Loveton Circle
Sparks, MD 21152 USA

ATCC is a registered trademark of the American Type Culture Collection.

© 2016 BD. BD, the BD Logo and all other trademarks are property of Becton, Dickinson and Company.