## NAMJENA

BD BBL DrySlide Oxidase (BD BBL DrySlide oksidaza) namijenjena je bakterijskoj reakciji na oksidazu.

## SAŽETAK I OBJAŠNJENJE

Test oksidaze kvalitativan je postupak za određivanje prisutnosti, odnosno odsutnosti aktivnosti oksidaze citokroma c u bakterijama. Aktivnost ovisi o prisutnosti međustaničnog citokromskog sustava oksidaze, koji katalizira oksidaciju citokroma c pomoću molekularnog kisika, koji zatim služi kao konačni akceptor elektrona u sustavu prijenosa elektrona organizma.<sup>1</sup>

Organizmi koji sadrže citokrom c u respiratornom lancu pozitivni su na oksidazu i reagens boje ljubičasto. Organizmi koji ne sadrže citokrom c u respiratornom lancu ne oksidiraju reagens pa on ostaje bez boje tijekom vremenskog ograničenja testa te su negativni na oksidazu.

## NAČELA POSTUPKA

N,N,N',N'-tetrametil-*p*-fenilenedijamin dihidroklorid aromatični je amin dimetiliran na dvije amino skupine. U reduciranu stanju reagens je bezbojan, a u oksidiranoj stanju tamnoljubičast.

U testu oksidaze oksidaza citokroma, koju proizvodi mikroorganizam, ne oksidira izravno reagens *p*-fenilenedijamin. Oksidira citokrom c, koji pak oksidira reagens te stvara spoj ljubičaste boje.

## REAGENSI

BD BBL DrySlide oksidaza stakalce je za jednokratnu upotrebu sa četiri reakcijska dijela te papirnim filtrom. Kao redukcijski agens i stabilizator sadrži N,N,N',N'-tetrametil-*p*-fenilenedijamin dihidroklorid i askorbinsku kiselinu.

### Upozorenja i mjere opreza:

Za *in vitro* dijagnostiku.

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231746 BD BBL DrySlide Oxidase

### LITTERATUR

1. Steel, K.J. 1961. The oxidase reaction as a taxonomic tool. *J. Gen. Microbiol.* 25: 297-306.
2. Tille, P.M. & Forbes, B.A. 2014. *Bailey & Scott's diagnostic microbiology*, 13th ed. St. Louis, MO.
3. Church, DL (ed.) 2007. *Aerobic Bacteriology*, Section 3.17.39.1-3. In Garcia, LS & Isenberg HD (ed.), *Clinical microbiology procedures handbook* (3rd ed.). American Society for Microbiology, Washington, D.C.
4. Latimer, GW (ed.) 2012. *Official methods of analysis of AOAC International* 19th ed. AOAC International, Gaithersburg, MD.
5. FDA bacteriological analytical manual (BAM), 2009. Taken from: <https://www.fda.gov/Food/FoodScienceResearch/LaboratoryMethods/ucm071363.htm>
6. Salfinger, Y & Tortorello ML (ed.), 2015. *Compendium of methods for the microbiological examination of foods*, 5th ed. American Public Health Association, Washington, D.C.
7. Henning, DR, et al., 2012. *Pathogens in milk and milk products*. In Wehr, HM & Frank, JF (ed.) *Standard methods for the examination of dairy products*, 17th ed. American Public Health Association, Washington, D.C.
8. Jorgensen, J.H., Pfaller, M.A., Carroll, K.C., 2015. *Manual of clinical microbiology*, 11th ed. American Society for Microbiology (ASM), Washington, D.C.
9. MacFaddin, J.F. 1980. *Biochemical tests for the identification of medical bacteria*, 2nd ed. Williams and Wilkins, Baltimore.

Tehnički servis i podrška: obratite se lokalnom predstavniku tvrtke BD ili posjetite [www.bd.com](http://www.bd.com).

Manufacturer / Производител / Výrobce / Fabrikant / Hersteller / Κατασκευαστής / Fabricante / Tootja / Fabricant / Proizvođač / Gyártó / Fabricante / Аткарушы / 제조업체 / Gamintojas / Ražotājs / Tilvirker / Producent / Producător / Производител / Výrobca / Proizvođač / Tillverkare / Üretici / Виробник / 生产厂商

Use by / Исполняйте до / Spottføjte do / Brug før / Verwendbar bis / Χρήση έως / Usar antes de / Date de péremption / 사용 기한 / Употребити до / Felhasználhatóság dátuma / Usare entro / Дейин пайдаланура / Naudokite iki / Izljetot līdz / Houdbaar tot / Brukes for / Stosować do / Prazo de validade / A se utiliza până la / Исползовать до / Použite do / Употребити до / Använd före / Son kullanna tarihi / Використати до / 使用截止日期

YYYY-MM-DD / YYYY-MM (MM = end of month)  
 ГГГГ-ММ-ДД / ГГГГ-ММ (ММ = края на месеца)  
 RRRR-MM-DD / RRRR-MM (MM = koniec miesiąca)  
 AAAA-MM-DD / AAAA-MM (MM = slutning af måned)  
 JJJJ-MM-TT / JJJJ-MM (MM = Monatsende)  
 EEEE-MM-HH / EEEE-MM (MM = τέλος του μήνα)  
 AAAA-MM-DD / AAAA-MM (MM = fin del mes)  
 AAAA-KK-PP / AAAA-KK (KK = kuu lõpp)  
 AAAA-MM-JJ / AAAA-MM (MM = fin du mois)  
 GGGG-MM-DD / GGGG-MM (MM = kraj mjeseca)  
 ÉÉÉÉ-HH-NN / ÉÉÉÉ-HH (HH = hónap utolsó napja)  
 AAAA-MM-GG / AAAA-MM (MM = fine mese)  
 ЖЖЖЖ-АА-КК / ЖЖЖЖ-АА (АА = айдың соңы)  
 YYYY-MM-DD/YYYY-MM (MM = 월말)  
 MMMM-MM-DD / MMMM-MM (MM = mēnesio pabaiga)  
 GGGG-MM-DD/GGGG-MM (MM = mēneša beigas)  
 JJJJ-MM-DD / JJJJ-MM (MM = einde maand)  
 AAAA-MM-DD / AAAA-MM (MM = slutten av måneden)  
 RRRR-MM-DD / RRRR-MM (MM = koniec miesiąca)  
 AAAA-MM-DD / AAAA-MM (MM = fim do mês)  
 AAAA-LL-ZZ / AAAA-LL (LL = sfârșitul lunii)  
 ГГГГ-ММ-ДД / ГГГГ-ММ (ММ = конец месяца)  
 RRRR-MM-DD / RRRR-MM (MM = koniec miesiąca)  
 GGGG-MM-DD / GGGG-MM (MM = kraj meseca)  
 AAAA-MM-DD / AAAA-MM (MM = slutet av månaden)  
 YYYY-AA-GG / YYYY-AA (AA = ayın sonu)  
 PPPP-MM-DD / PPPP-MM (MM = кінець місяця)  
 YYYY-MM-DD / YYYY-MM (MM = 月末)

Catalog number / Каталоген номер / Katalogové číslo / Katalognummer / Αριθμός καταλόγου / Número de catálogo / Katalooginumber / Numéro catalogue / Kataloški broj / Katalogszám / Numero di catalogo / Каталог номери / 카탈로그 번호 / Katalogo / numeris / Kataloga numurs / Catalogus nummer / Numer katalogowy / Număr de catalog / Номер по каталогу / Katalogové číslo / Kataloški broj / Katalog numarası / Номер за каталогом / 目录号

Authorized Representative in the European Community / Оторизиран представител в Европейската общност / Autorizovaný zástupce pro Evropském společenství / Autoriseret representant i De Europæiske Fællesskaber / Autoriserter Vertreter in der Europäischen Gemeinschaft / Εξουσιοδοτημένος αντιπρόσωπος στην Ευρωπαϊκή Κοινότητα / Representante autorizado en la Comunidad Europea / Volitatud esindaja Euroopa Nõukogus / Représentant autorisé pour la Communauté européenne / Autorizirani predstavnik u Europskoj uniji / Meghatalmazott képviselő az Európai Közösségben / Rappresentante autorizzato nella Comunità Europea / Европа қауымдастығындағы уәкілетті өкіл / 유럽 공동체의 위임 대표 / Igalotasis atstovas Europos Bendrijoje / Pilnvarotais pārstāvis Eiropas Kopienā / Bevoegde vertegenwoordiger in de Europese Gemeenschap / Autoriseret representant i EU / Autoryzowane przedstawicielstwo we Wspólnocie Europejskiej / Representante autorizado na Comunidade Europeia / Reprezentantul autorizat pentru Comunitatea Europeană / Уполномоченный представитель в Европейском сообществе / Autorizovaný zástupce v Európskom spoločenstve / Autorizovano predstavništvo u Evropskoj uniji / Auktoriserad representant i Europeiska gemenskapen / Автура Topluluğu Yetkilil Temsilcisi / Уповноважений представител в / 欧洲共同体授权代表

In Vitro Diagnostic Medical Device / Медицински уред за диагностика ин витро / Lékařské zařízení určené pro diagnostiku in vitro / In vitro diagnostisk medicinsk anordning / Medizinisches In-vitro-Diagnostikum / In vitro διαγνωστική ιατρική συσκευή / Dispositivo médico para diagnóstico in vitro / In vitro diagnostika meditsiinaparatuur / Dispositif médical de diagnostic in vitro / Medicinska pomagala za In Vitro Dijagnostiku / In vitro diagnostiztikai orvosi eszköz / Dispositivo medicale per diagnostica in vitro / Жасанды жағдайда жүргізетін медициналық диагностика аспабы / In Vitro Diagnostic 의료 기기 / In vitro diagnostikos prietaisais / Medicinas ierīces, ko lieto in vitro diagnostikā / Medisch hulpmiddel voor in-vitro diagnostiek / In vitro diagnostisk medicinsk utstyr / Urządzenie medyczne do diagnostyki in vitro / Dispositivo médico para diagnóstico in vitro / Dispozitiv medical pentru diagnostic in vitro / Медицинский прибор для диагностики ин витро / Medicinska pomůcka na diagnostiku in vitro / Medicinski uređaj za in vitro dijagnostiku / Medicinteknisk produkt för in vitro-diagnostik / In Vitro Diagnostik Tibbi Cihaz / Медицинский прибор для диагностики in vitro / 体外诊断医疗设备

Temperature limitation / Температурни ограничения / Teplotní omezení / Temperaturbegrænsning / Temperaturbegrensning / Περιορισμοί θερμοκρασίας / Limitación de temperatura / Temperatuuri piirang / Limites de température / Dozvoljena temperatura / Hőmérsékleti határ / Limiti di temperatura / Температураны шекте / 온도 제한 / Laikymo temperatūra / Temperatūras ierobežojumi / Temperaturulimiet / Temperaturbegrensning / Ograniczenie temperatury / Limites de temperatura / Limite de temperatură / Ограничение температуры / Ohraničenje teploty / Ograničenje temperature / Temperaturgräns / Sıcaklık sınırlaması / Обмеження температури / 温度限制

Batch Code (Lot) / Код на партидата / Kód (číslo) šarže / Batch-kode (lot) / Batch-Code (Charge) / Κωδικός παρτίδας (παρτίδα) / Código de lote (lote) / Partii kood / Numéro de lot / Lot (kod) / Tétel száma (Lot) / Codice batch (lotto) / Топама коды / 배치 코드(로트) / Partijos numeris (LOT) / Partijas kods (laidiens) / Lot number / Batch-kode (parti) / Kod partii (seria) / Código do lote / Cod de serie (lot) / Код партии (lot) / Kód série (šarža) / Kod serije / Partinummer (Lot) / Parti Kodu (Lot) / Код партии / 批号 (亚批)

Contains sufficient for <n> tests / Съдържанието е достатъчно за <n> теста / Dostatečné množství pro <n> testů / Indeholder tilstrækkeligt til <n> tests / Ausreichend für <n> Tests / Περιέχει επαρκή ποσότητα για <n> εξετάσεις / Contenido suficiente para <n> pruebas / Küllaldane <n> testide jaoks / Contenu suffisant pour <n> tests / Sadržaj za <n> testova / <n> teszthez elegendő / Contenuto sufficiente per <n> test / <n> тесттеги үшін жеткілікті / <n> 테스트가 충분한 포함됨 / Pakankamas kiekis atlikti <n> testų / Satur pietiekami <n> pārbaudēm / Inhoud voldoende voor "n" testen / Innholder tilstrekkelig til <n> tester / Zawiera ilość wystarczającą do <n> testów / Conteúdo suficiente para <n> testes / Conținut suficient pentru <n> teste / Достаточно для <n> тестов(а) / Obsah vystačí na <n> testov / Sadržaj dovoljan za <n> testova / Innehåller tillräckligt för <n> analyser / <n> test için yeterli malzeme içerir / Вистачить для аналізів: <n> / 足够进行 <n> 次检测

Consult Instructions for Use / Направете справка в инструкциите за употреба / Prostudujte pokyny k použití / Se brugsanvisningen / Gebrauchsanweisung beachten / Συμβουλευτείτε τις οδηγίες χρήσης / Consultar las instrucciones de uso / Lugeda kasutusjuhendit / Consulter la notice d'emploi / Koristi upute za upotrebu / Olvassa el a használati utasítást / Consultare le istruzioni per l'uso / Пайдалану нұсқаулығымен танысып алыңыз / 사용 지침 참조 / Skaitykite naudojimo instrukcijas / Skatīt lietošanas pamācību / Raadpleeg de gebruiksaanwijzing / Se i bruksanvisningen / Zobacz instrukcja użytkowania / Consultar as instruções de utilização / Consultați instrucțiunile de utilizare / См. руководство по эксплуатации / Pozri Pokyny na používanie / Pogledajte uputstvo za upotrebu / Se bruksanvisningen / Kullanım Talimatları'na başvurun / Див. інструкції з використання / 请参阅使用说明

Becton, Dickinson and Company  
 7 Loveton Circle  
 Sparks, MD 21152 USA

Benex Limited  
 Pottery Road, Dun Laoghaire  
 Co. Dublin, Ireland

**Australian Sponsor:**

Becton Dickinson Pty Ltd.  
 4 Research Park Drive  
 Macquarie University Research Park  
 North Ryde, NSW 2113  
 Australia

ATCC is a trademark of the American Type Culture Collection.

DrySlide is a trademark of Difco Laboratories, a subsidiary of Becton, Dickinson and Company.

© 2018 BD. BD, the BD Logo and all other trademarks are property of Becton, Dickinson and Company.