# BD BBL Taxo A Discs for Differentiation of Group A Streptococci

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## REFERENCE

1. Maxted, W.R. 1953. The use of bacitracin for identifying group A haemolytic streptococci. *J. Clin. Pathol.* 6:224-226.
2. Levinson, M.L., and P.F. Frank. 1955. Differentiation of group A from other beta-hemolytic streptococci with bacitracin. *J. Bacteriol.* 69:284-287.
3. Streamer, C.W., P.M. Williams, W.L. Wang, R.S. Johnson, C.D. McGuire, I.J. Abelow, and R.J. Glaser. 1962. Identification of group A streptococci. Bacitracin disc and fluorescent antibody techniques compared with the Lancefield precipitin method. *Am. J. Dis. Child.* 104:157-160.
4. Petran, E.I. 1964. Comparison of the fluorescent antibody and the bacitracin disk methods for the identification of group A streptococci. *Am. J. Clin. Pathol.* 41:224-226.
5. Moffet, H.L., H.G. Cramblett, and J.P. Black. 1964. Group A streptococcal infections in a children's home. 1. Evaluation of practical bacteriologic methods. *Pediatrics* 32:5-10.
6. Chitwood, L.A., M.B. Jennings, and H.D. Riley, Jr. 1969. Time, cost, and efficacy study of identifying group A streptococci with commercially available reagents. *Appl. Microbiol.* 18:193-197.
7. Evans, G.L., and T.E. O'Neill. 1984. Development of an improved selective medium for the isolation of group A streptococci from throat cultures, abstr. C-136, p. 259. Abstr. 84th Annu. Meet. Am. Soc. Microbiol. 1984.
8. Facklam, R.R. and R.B. Cary. 1985. Streptococci and aerococci, p. 154-175. In E.H. Lennette, A. Balows, W.J. Hausler, Jr., and H.J. Shadomy (ed.), *Manual of clinical microbiology*, 4th ed. Am. Soc. Microbiol., Washington, D.C.
9. Baron, E.J., and S.M. Finegold. 1990. *Bailey & Scott's diagnostic microbiology*, 8th ed. The C.V. Mosby Company, St. Louis.
10. Facklam, R.R., J.F. Padula, L.G. Thacker, E.C. Wartham, and B.J. Sconyers. 1974. Presumptive identification of group A, B and D streptococci. *Appl. Microbiol.* 27:107-113.

Tehnički servis i podrška na BD Diagnostics: obratite se lokalnom predstavniku tvrtke BD ili posjetite [www.bd.com/ds](http://www.bd.com/ds).

Manufacturer / Производител / Výrobce / Fabrikant / Hersteller / Κατασκευαστής / Fabricante / Tootja / Fabricant / Proizvođač / Gyártó / Fabbicante / Аткарушы / Gamintojas / Ražotājs / Tilvirker / Producent / Producător / Производитель / Výrobca / Proizvođač / Tilverkare / Üretici / Виробник

Use by / Используйте до / Spotføjebjtte do / Brug før / Verwendbar bis / Χρήση έως / Usar antes de / Kasutada enne / Date de péremption / Upotrijebiti do / Felhasználhatóság dátuma / Usare entro / Дейін пайдалануға / Naudokite iki / Izlietot līdz / Houdbaar tot / Brukes for / Stosować do / Prazo de validade / A se utiliza până la / Исползовать до / Použite do / Upotrebiti do / Använd före / Son kullanna tarini / Використати долине

YYYY-MM-DD / YYYY-MM (MM = end of month)  
 ГГГГ-ММ-ДД / ГГГГ-ММ (ММ = края на месеца)  
 RRRR-MM-DD / RRRR-MM (MM = konec měsíce)  
 AAAA-MM-DD / AAAA-MM (MM = slutning af måned)  
 JJJJ-MM-TT / JJJJ-MM (MM = Monatsende)  
 EEEE-MM-HH / EEEE-MM (MM = τέλος του μήνα)  
 AAAA-MM-DD / AAAA-MM (MM = fin del mes)  
 AAAA-KK-PP / AAAA-KK (KK = kuu lõpp)  
 AAAA-MM-JJ / AAAA-MM (MM = fin du mois)  
 GGGG-MM-DD / GGGG-MM (MM = kraj mjeseca)  
 ÉÉÉÉ-HH-NN / ÉÉÉÉ-HH (HH = hónap utolsó napja)  
 AAAA-MM-GG / AAAA-MM (MM = fine mese)  
 ЖӨЖӨК-АА-КК / ЖӨЖӨК-АА / (АА = айдың соңы)  
 MMMM-MM-DD / MMMM-MM (MM = mēnesio pabaiga)  
 GGGG-MM-DD/GGGG-MM (MM = mēneša beigas)  
 JJJJ-MM-DD / JJJJ-MM (MM = einde maand)  
 AAAA-MM-DD / AAAA-MM (MM = slutten av månaden)  
 RRRR-MM-DD / RRRR-MM (MM = koniec miesiąca)  
 AAAA-MM-DD / AAAA-MM (MM = fim do mês)  
 AAAA-LL-ZZ / AAAA-LL (LL = sfârșitul lunii)  
 ГГГГ-ММ-ДД / ГГГГ-ММ (ММ = конец месяца)  
 RRRR-MM-DD / RRRR-MM (MM = koniec mesiacu)  
 GGGG-MM-DD / GGGG-MM (MM = kraj meseca)  
 AAAA-MM-DD / AAAA-MM (MM = slutet av månaden)  
 YYYY-AA-GG / YYYY-AA (AA = айн солу)  
 PPPP-MM-DD / PPPP-MM (MM = кінець місяця)

Catalog number / Каталоген номер / Katalogové číslo / Katalognummer / Αριθμός καταλόγου / Número de catálogo / Katalognummer / Numéro catalogue / Kataloški broj / Katalogszám / Numero di catalogo / Каталог номери / Katalogo numeris / Katalogo numurs / Catalogus nummer / Numer katalogowy / Număr de catalog / Номер по каталору / Katalogové číslo / Kataloški broj / Katalog numarası / Номер за каталогом

Authorized Representative in the European Community / Оторизиран представител в Европейската общност / Autorizovaný zástupce pro Evropském společenství / Autoriseret repræsentant i De Europæiske Fællesskaber / Autorisierter Vertreter in der Europäischen Gemeinschaft / Εξουσιοδοτημένος αντιπρόσωπος στην Ευρωπαϊκή Κοινότητα / Representante autorizado en la Comunidad Europea / Volitatud esindaja Euroopa Nõukogus / Représentant autorisé pour la Communauté européenne / Autorizuirani predstavnik u Europskoj uniji / Meghatalmazott képviselő az Európai Közösségben / Rappresentante autorizzato nella Comunità Europea / Европа қауымдастығындағы уәкілетті өкіл / Įgaliojasis atstovas Europos Bendrijoje / Pilnvarotais pārstāvis Eiropas Kopienā / Bevoegdte vertegenwoordiger in de Europese Gemeenschap / Autoriseret representant i EU / Autoryzowane przedstawicielstwo we Wspólnocie Europejskiej / Representante autorizado na Comunidade Europeia / Reprezentantul autorizat pentru Comunitatea Europeană / Уполномоченный представитель в Европейском сообществе / Autorizovaný zástupca v Európskom spoločenstve / Autorizovano predstavništvo u Evropskoj uniji / Auktoriserad representant i Europeiska gemenskapen / Avrupa Topluluğu Yetkilii Temsilcisi / Уповноважений представник у країнах ЄС

In vitro Diagnostic Medical Device / Медицински уред за диагностика ин vitro / Lékařské zařízení určené pro diagnostiku in vitro / In vitro diagnostik medicinsk anordning / Medizinisches In-vitro-Diagnostikum / In vitro διαγνωστική ιατρική συσκευή / Dispositivo médico para diagnóstico in vitro / In vitro diagnostika meditsiiniparatuur / Dispositif médical de diagnostic in vitro / Medicinska pomagala za in vitro Dijagnostiku / In vitro diagnostikai orvosi eszköz / Dispositivo medicale per diagnostica in vitro / Жасанды жағдайда жүргізілетін медициналық диагностика аспабы / In vitro diagnostikos prietaisas / Medicīnas ierīces, ko lieto in vitro diagnostikā / Medisch hulpmiddel voor in-vitro diagnostiek / In vitro diagnostisk medisinsk utstyr / Urządzenie medyczne do diagnostyki in vitro / Dispositivo médico para diagnóstico in vitro / Dispositiv medical pentru diagnostic in vitro / Медицинский прибор для диагностики in vitro / Medicinska pomôcka na diagnostiku in vitro / Medicinski uređaj za in vitro dijagnostiku / Medicinteknisk produkt för in vitro-diagnostik / In Vitro Dijagnostik Tibbi Cihaz / Медичний пристрій для діагностики in vitro

Temperature limitation / Температурни ограничения / Teplotní omezení / Temperaturbegrænsning / Temperaturbegrenzung / Περιορισμοί θερμοκρασίας / Limitación de temperatura / Temperatuuri piirang / Limites de température / Dozvoljena temperatura / Hőmérsékleti határ / Limiti di temperatura / Температураны шектеу / Laikymo temperatūra / Temperatūras ierobežojumi / Temperaturlimit / Temperaturbegrensnng / Ograniczenie temperatury / Limites de temperatura / Limite de temperatură / Ограничение температуры / Ohraničenje teploty / Ograničenje temperature / Temperaturgräns / Sicaklık sınırlaması / Обмеження температури

Batch Code (Lot) / Код на партидата / Kód (číslo) šarže / Batch-kode (lot) / Batch-Code (Charge) / Κωδικός παρτίδας (παρτίδα) / Código de lote (lote) / Partii kood / Numéro de lot / Lot (kod) / Tétel száma (Lot) / Codice batch (lotto) / Топтама коды / Partijos numeris (LOT) / Partijas kods (laidiens) / Lot nummer / Batch-kode (parti) / Kod partii (seria) / Código do lote / Cod de serie (Lot) / Код партии (лот) / Kód série (šarža) / Kod serije / Partinummer (Lot) / Parti Kodu (Lot) / Код партії

Consult Instructions for Use / Направете справка в инструкциите за употреба / Prostudujte pokyny k použití / Se brugsanvisningen / Gebrauchsanweisung beachten / Συμβουλευτείτε τις οδηγίες χρήσης / Consultar las instrucciones de uso / Lugeda kasutusjuhendit / Consulter la notice d'emploi / Koristi upute za upotrebu / Olvassa el a használati utasítást / Consultare le istruzioni per l'uso / Пайдалану нұсқаулығымен танысып алыңыз / Skaitykite naudojimo instrukcijas / Skatīt lietošanas pamācību / Raadpleeg de gebruiksaanwijzing / Se i bruksanvisningen / Zobacz instrukcja użytkowania / Consultar as instruções de utilização / Consultați instrucțiunile de utilizare / См. руководство по эксплуатации / Pozri Pokyny na používanie / Pogledajte uputstvo za upotrebu / Se bruksanvisningen / Kullanım Talimatları'na başvurun / Див. інструкції з використання

Becton, Dickinson and Company  
 7 Loveton Circle  
 Sparks, MD 21152 USA

Benex Limited  
 Pottery Road, Dun Laoghaire  
 Co. Dublin, Ireland

**Australian Sponsor:**

Becton Dickinson Pty Ltd.  
 4 Research Park Drive  
 Macquarie University Research Park  
 North Ryde, NSW 2113  
 Australia

ATCC is a trademark of the American Type Culture Collection.  
 BD, BD Logo, and all other trademarks are property of Becton, Dickinson and Company. © 2015 BD.